

Access Mobility Center

7620 Eastman Avenue
Midland, MI 48642
989-633-6100

For a Service Appointment

Lift Inspection and Maintenance Checklist

- Inspect and lube door hinges & friction stops on doors around lift
- Check pendant mount wires and connections
- Deploy lift to ground position, remove cover and check fluid levels
- Inspect all wiring connectors under pump housing cover
- Inspect all limit switches
- Inspect manual pump for proper operation
- Inspection of all hydraulic hoses for leaks, cracks or imperfections
- Inspection of all hydraulic vent hoses
- Inspection of stow latch operation and clean
- Inspection of all deploying springs
- Inspect, clean and lube bridge plate hinges
- Inspect bridge plate actuator arms and cables
- Inspect hydraulic cylinders for leaks or loose fitting parts
- Inspect safety strap and the operation of safety strap limit switches
- Inspection of safety stow weight restriction
- Inspection of lift base and arms for any stress marks or cracks
- Check all bolts for correct torques
- Inspection of all retaining clip rings
- Inspect roll-stop mechanism for proper operation
- Inspect roll-stop hoses and cylinder for no imperfections
- Inspect, clean and lubricate all moving parts (arms and pivot points)
- Inspect battery terminals for corrosion or improper connections
- Inspect power supply cable from battery to the lift
- Inspect all connectors on circuit breaker
- *Correct any minor repairs or adjustments @ n/c. Only to prevent our customers from further service appointments or other inconveniences.*